NSK Varios Tips

VarioSurg Tips

OPERATION MANUAL

CE 0197 MADE IN JAPAN

Tip Power Guide

NAKANISHI INC. www.nsk-dental.com
700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK Europe GmbH
Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

Visit our website

Specifications are subject to change without notice.

2021-11-10 CAGB1117 00517

EN This operation manual describes the procedure of mounting/removing and reprocessing the tip. Carefully read the operation manual of the Ultrasonic System and the tips for other procedures.

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EN • Only hold the tip at the base where it screws on to the handpiece. Otherwise there is a risk of injury to your hand or tip damage. • When attaching the tip, always wear sterilized gloves, and use a tip, handpiece, and tip replacement wrench that are all sterilized. • Make sure you don't twist the handpiece cord when tightening the tip.

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EN • Too tight mounting with the E-Tip wrench/VS Tip wrench could cause breakage. Too loose mounting could reduce the power to the tip. • When mounting the tip to V10/V10-S, be careful not to excessively tighten the tip holder screw (*) as the holder screw may break. Tighten the screw gently by hand before cleaning and sterilization. • Appropriate vibration is not guaranteed unless the tip is inserted up to the dent mark (*2).

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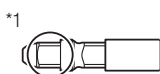
EN • Refer to the operation manual for the ultrasonic system that the handpiece and tip are to be connected to set the power level.

• Refer to the table 1 of this manual which defines the maximum power level of the tips according to the ultrasonic system which is being used.

1) Set the power of the ultrasonic system at the lower end of the permitted range and increase gradually depending on the patient condition.

2) Adjust the irrigation flow in order for the coolant water to reach the tip end.

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EN • During its operation, the whole tip vibrates due to ultrasonic waves. Do not bring any part of the tip into contact with soft tissue, gums or skin. Regardless of the use of coolant water the tip in vibration could cause a burn injury.

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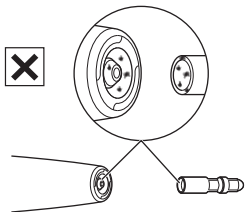
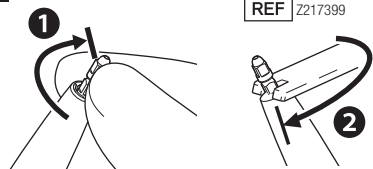
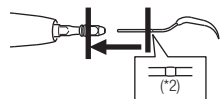
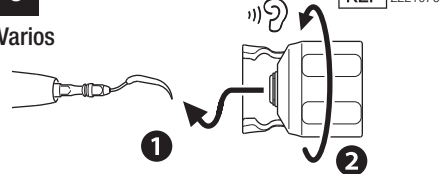
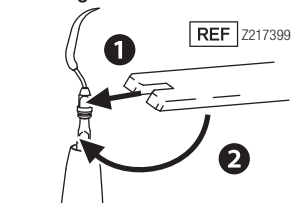
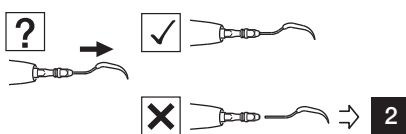
EN Failure to properly maintain as below, the tip may cause infection, product failure, overheating leading to burn injuries.
 - Be sure to clean the tip immediately (within 30 minutes) after use to remove residue.

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EN • Before using an alcoholic product for internal disinfection, the Tip must first be cleaned with a non-protein-fixing product (e.g. ALPRO WL-clean). • Use only cleaning/disinfectant products listed in this manual. • Follow any additional local directives, standards and guidelines for cleaning, disinfection, and sterilization.

• Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water; super acidic water), strong acidic and strong alkaline liquid medicine, solvent containing chlorine, benzene, or thinner. • Use clean and dry compressed air for cleaning. • Tips must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning, disinfecting and drying cycle is complete to prevent corrosion. • After using washer-disinfector, dry the product completely. The remaining moisture causes internal corrosion etc. • Follow local rules, regulations and guidelines regarding the reprocessing of devices. • Do not autoclave the tip and the tip wrench with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments. • Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product. • Be sure to use sterilizers that can perform sterilization up to 135°C. In some sterilizers, the chamber temperature may exceed 135°C. Do not use these sterilizers as failure of the product could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures. • Clean the tip and the tip wrench prior to disinfection. If blood remains on the internal surface it can become clotted and cause product failure. • Immediately after sterilization is complete (within 1 hour), remove the tip and the tip wrench from the sterilizer. Failure to do so may cause corrosion.

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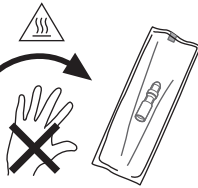
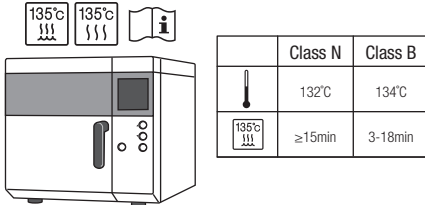
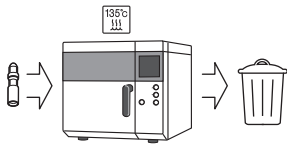
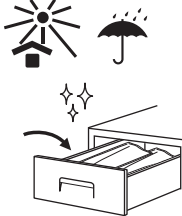
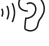
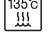
EN • This tip is specially designed for the Ultrasonic System Varios970, Varios570, Varios370, Varios170, Varios750, Varios560, Varios350 and Varios150 by NSK. Do not use with other systems. • This tip is specially designed for the Ultrasonic System VarioSurg and VarioSurg 3 by NSK. Do not use with other systems. • Be sure to always carefully read the operation manual for the Ultrasonic System before use to understand the usage and performance of the instrument. • Be sure to always use with sufficient coolant water. Insufficient coolant water could cause a burn of the affected area or the overheating of the instrument. • Be sure to always use the tip at a setting not exceeding the maximum power level. If the tip is used in excess of the maximum power, the tip could break or excess cutting could occur. • Do not touch the top part of the tip to avoid an injury.

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EN • This product must only be used by a qualified personnel in accordance with the indications for use and the directions for use stated in the operation manual of the Ultrasonic System it is attached to and this operation manual. Do not use it for any other purpose. • Operators and all others in the area must wear eye protection and a mask when operating this tip. • Prior to clinical use, inspect the tip outside the patient's oral cavity. Check for vibration, noise and overheating. If any abnormalities are found, stop using the tip immediately and contact your Authorized NSK Dealer. • The tip is a consumable item and breaks with use. If it is worn, abnormal noise, weak or no vibration, or damage may occur. Replace the tip with a new one when it is worn. Also, the tip can break due to metal fatigue. Replace with a new tip on a regular basis. • The tip may be discolored by repeated reprocessing. If the tip is discolored significantly, replace it with a new one. • Do not grind or bend the tip. It may break or may not vibrate during use. • Keep away from patients with cardiac pacemakers. • The tip is delivered in a non-sterile condition and must be autoclaved prior to use. • For irrigation, avoid use of ozonated water, oxidizing water (strongly acidic water, superacidic water, electrolyzed water, etc.), or hypochlorous solution. Doing so may lead to clogging, corrosion of metal parts, etc. • Perform suction while using the tip, since the tip may break depending on its use condition. • Even if the power level is correctly set for the Tip in use, application of excessive pressure to the Tip may damage it. • During vibration, do not hold the tip nor make contact with the tip for an extended time. • If there are abnormalities in water supply during operation, stop using the tip immediately and replace it with a new one. • If the tip cracks or breaks during use, immediately stop using the tip, and replace with a new tip. Operation with a damaged tip could injure the gums, periodontal tissue, or other locations. • Do not apply more pressure than required to the tip end. This could cause the tip to break or bend. • Start with a low power setting within the power range, observing the patient's reaction and according to the treatment plan. • Before use, check that the tip is firmly attached to the handpiece. • Refer to the operation manual of the ultrasonic system for EMC information. • Report all serious events related to the product immediately to the manufacturer and responsible national authority.

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