



EN Table 1 – Recommended Power setting
 DE Tabelle 1 – Empfohlene eingestellte Leistung
 FR Tableau 1 – Réglages de puissance recommandés
 ES Tabla 1 – Ajuste de potencia recomendada
 IT Tabella 1 – Impostazione di potenza raccomandata
 PT Tabela 1 – Definição de potência recomendada

Model		Mode	
SG1			80% B
SG1A			80% B
H-SG1			150% B
SG2R			80%
SG2L			80%
SG3			80%
SG4			80%
SG5			80%
SG6D		✓	80%
SG7D		✓	50%
SG8			80% B
SG8A			80% B
H-SG8		((S))	150% B
H-SG8R			150% A
H-SG8L			150% A
SG9			50%
SG10			50%
SG11			50%
SG14R			80%
SG14L			80%
SG17			80%
SG18R			50%
SG18L			50%
SG19			80% B
SG30			50%
SG58			80% B
SG68			80% A
SG15A		✓	50% C
SG15B		✓	50% C
SG15C		✓	50% D
SG15D		✓	50% D
SG16A		✓	50% C
SG16B		✓	50% C
SCL1*			50% E
SCL1D*		✓	80% E
SCL2D*		✓	80% E
SCL3*			50% E
SCL3D*		✓	80% E
SCL4D*		✓	80% E
SCL5*			50% E
SCL5D*		✓	80% E
E30RD-S		✓	50%
E30LD-S		✓	50%
E31D-S		✓	50%
E32D-S		✓	50%
G1-S		((S))	50%
P20-S			100%
P25R-S		(P)	100%
P25L-S			100%


EN *Refer to OM-DZ0916MA for how to use the Tips.
 DE *Informations zur Verwendung der Aufsätze finden Sie unter OM-DZ0916MA.
 FR *Réferez-vous au document OM-DZ0916MA pour connaître le mode d'utilisation des inserts.
 ES *Consulte OM-DZ0916MA para obtener información sobre cómo usar las puntas.
 IT *Consultare OM-DZ0916MA per informazioni sull'utilizzo delle punte.
 PT *Consulte OM-DZ0916MA sobre como utilizar as pontas.

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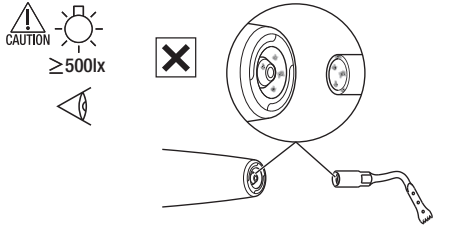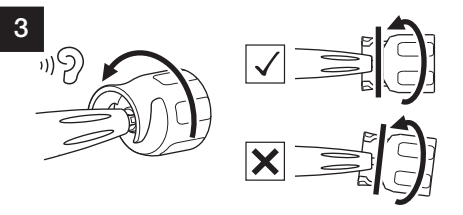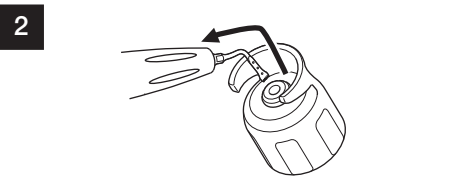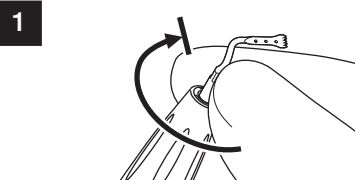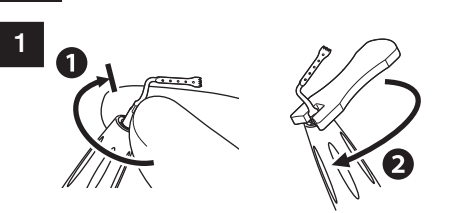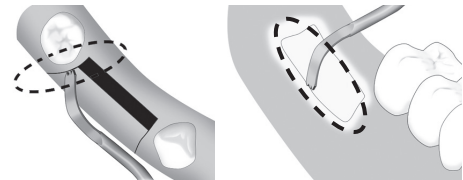
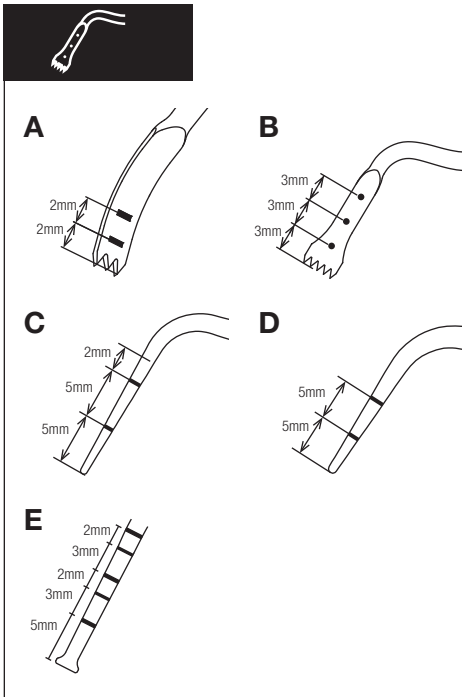
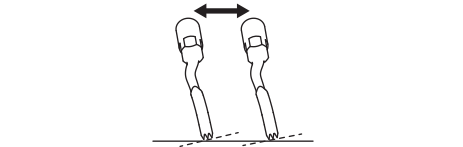
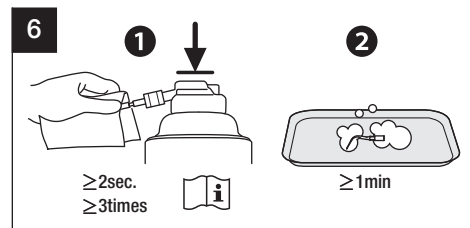
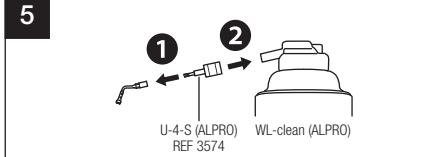
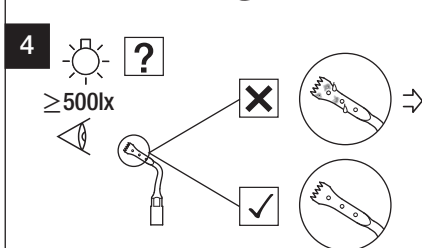
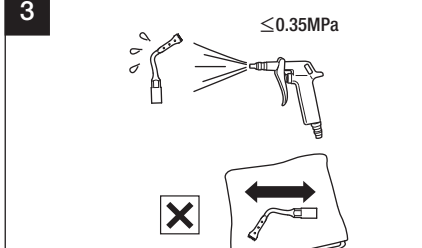
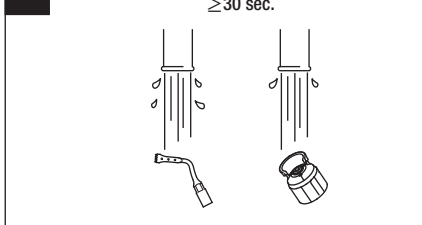
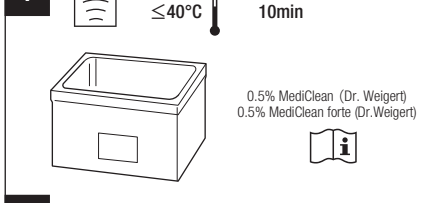
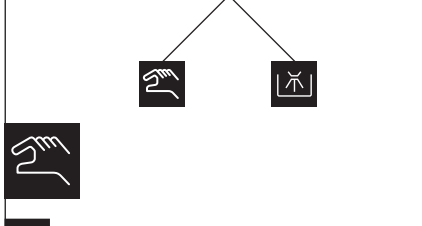
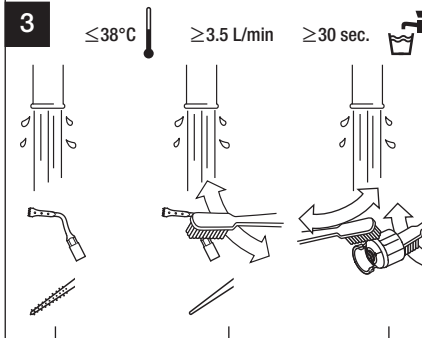
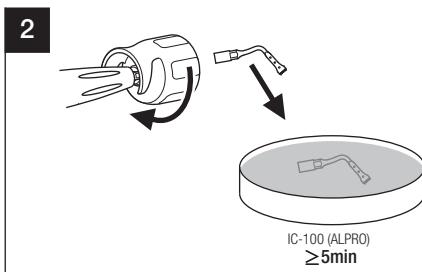
CAUTION



EN • Before using an alcoholic product for internal disinfection, the Tip must first be cleaned with a non-protein-fixing product (e.g. ALPRO WL-clean). • Clean the tip and the tip wrench prior to disinfection. If blood remains on the internal surface it can become clotted and cause product failure. • Use only cleaning disinfectant products listed in this manual. • Follow any additional local directives, standards and guidelines for cleaning, disinfection, and sterilization. • Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid medium, solvent containing chlorine, benzene, or thinner. • Use clean and dry compressed air for cleaning. • Tips must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning, disinfecting and drying cycle is complete to prevent corrosion. • After using washer-disinfector, dry the product completely. The remaining moisture causes internal corrosion etc. • Do not autoclave the tip and the tip wrench with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments. • Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product. • Immediately after sterilization is complete (within 1 hour), remove the tip and the tip wrench from the sterilizer. Failure to do so may cause corrosion.

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
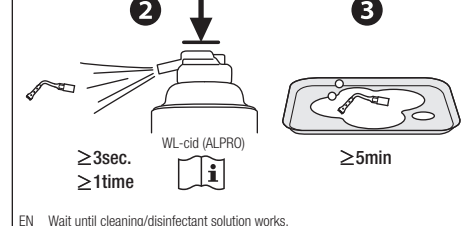
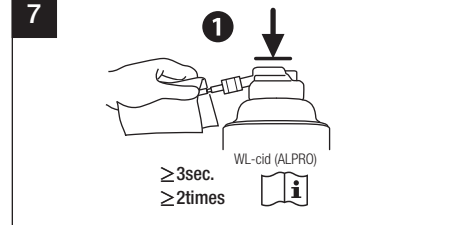
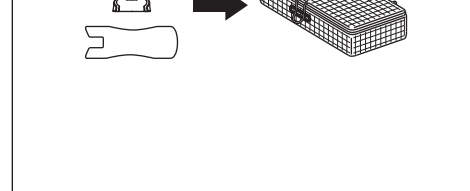
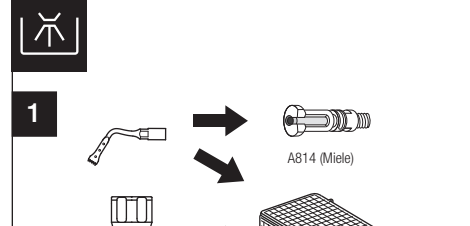
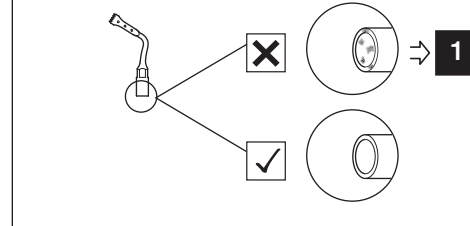
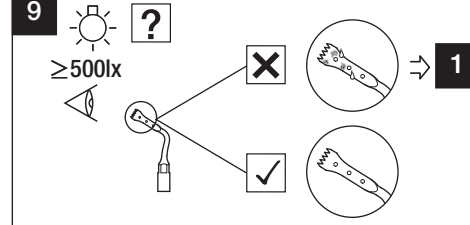
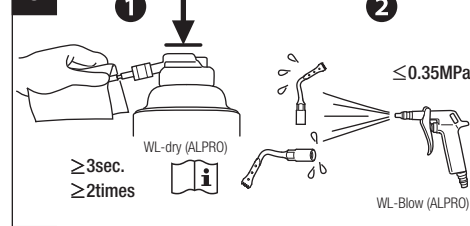
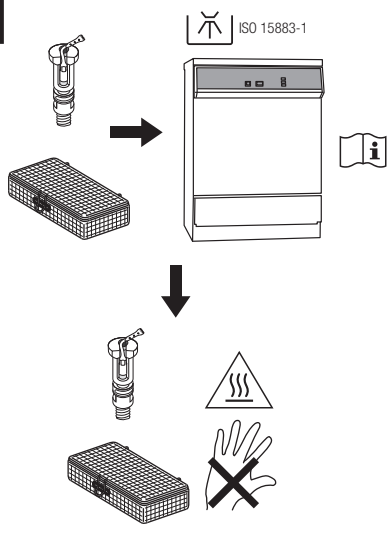
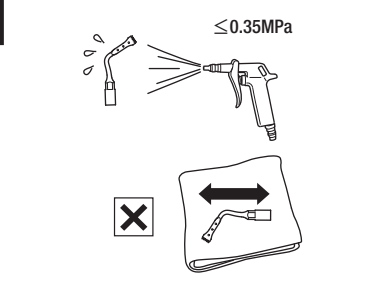
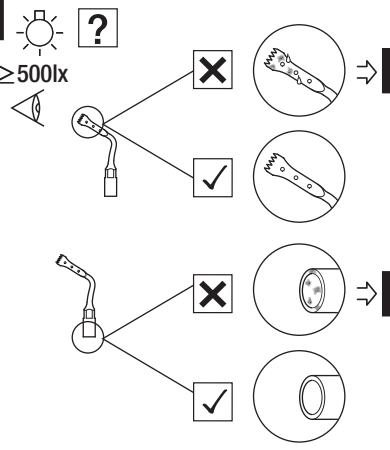
135°C

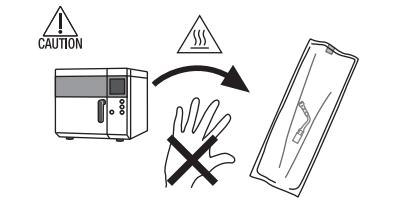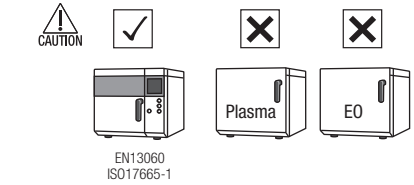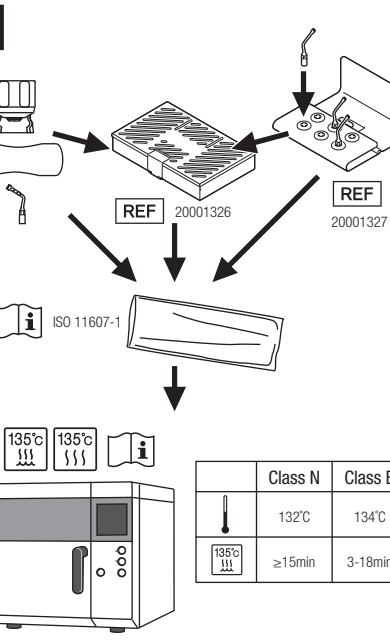1

EN Do not use sterilization pouches containing water-soluble adhesive components such as PVA (polyvinyl alcohol). Adhesive components washed out during sterilization may enter the product, causing problems such as malfunction, sticking and failure to operate correctly.

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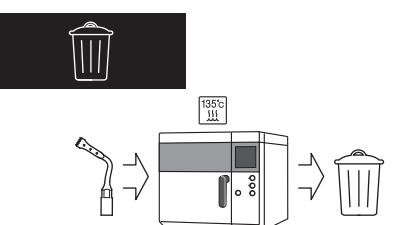
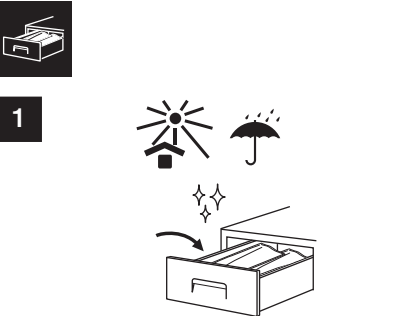
EN When your medical device has reached the end of its service life, it must be disposed of via a specialist collection, removal, recycling or destruction channel. Prior to do so, in order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, you need to confirm the equipment is decontaminated. Therefore contact your nearest NSK dealer, or one of the NSK branches to find out how to proceed. Refer to the link below:

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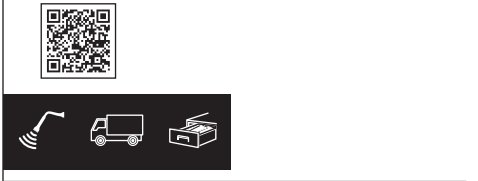
EN Intended purpose
The dental surgical device is intended for the following application(s):

- H-SG1, H-SG8, SG58, SG1, SG1A, SG2R, SG2L, SG3, SG4, SG5, SG6D, SG7D, SG8, SG8A, SG14R, SG14L, SG19, SG30, H-SG8R, H-SG8L, SG68: Cutting, removal, incision, and perforation of jawbones.
- SCL1D, SCL2D, SCL3D, SCL4D, SCL5D, SG15A, SG15B, SG15C, SG15D, SG16A, SG16B: Removal and perforation of jawbones.
- SG9, SG10, SG11, SCL1, SCL3, SCL5: Detachment of mucosal membranes.
- SG17, SG18R, SG18L: Detachment of periodontal membrane (as part of preparation for placement of dental implants).
- G1-S: Mainly used for removal of supragingival or marginal gingival accretions and deposits.
- E30RD-S, E30LD-S, E31D-S, E32D-S: Root canal preparation for retrograde endodontics (including removal of root canal filling materials).
- P20-S, P25R-S, P25L-S: Mainly used for removal of subgingival accretions and deposits.

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